

Hyten™ Elevated Toilet Seat

INSTALLATION REFERENCE GUIDE

Toilet seat adds 3" to your toilet bowl.

The Hyten Elevated toilet seat is easy to install and requires just a screwdriver. And with Quick-Release™ technology, you can remove the seat in just seconds for fast, convenient cleaning.

- 1 Place anchor hardware through the bolt holes by folding them vertically.



- 2 Turn the seat upside down, and place the hinge mounts into the two holes.



- 3 Turn the seat right side up, and position it over the toilet bowl, orienting the hinge mounts to align with the holes of the toilet bowl.



- 4 Insert screws through the hinge mounts into the Grip-Tight hardware and tighten with a screwdriver so the screws are secure within the mounting system.



- 5 Remove seat after tightening down screws to position the hinge caps over the metal hinge posts.



- 6 Place the hinge caps over the hinge posts to cover the installation hardware.



- 7 Position the seat over the hinge posts and push down, ensuring the seat securely locks into the hinge posts.



- 8 A properly installed seat will be positioned securely on the toilet bowl. To remove the seat for cleaning, simply push the Quick-Release button and remove seat.

